

19% EDTA Cream

Root Canal Cleaning and Shaping and Smear Layer Removal



Applications:

- ✓ Root canal preparation with files and reamers
- ✓ Presence of calcifications in the canal
- ✓ Narrow canals in posterior teeth
- ✓ Smear layer removal

Advantages:

- ✓ Removal of inorganic substances by chelating (EDTA-Ca²⁺)
- ✓ Easier filing with lubricating action
- ✓ Produces the cleanest canal wall after smear layer removal

Components:

- ✓ MD-ChelCream 5.4 mL × 2
- ✓ 10 Disposable tips
- ✓ Dimension: 195 × 70 × 30 (mm)
- ✓ Weight: 72g

Directions:

1. Deposit small amount in the canal orifice using the MD-ChelCream syringe.
2. Carefully pump into the canal with a fine file or reamer.
3. Re-apply MD-ChelCream when necessary or until cleaning and shaping is complete.
4. Rinse out MD-ChelCream by irrigation with sodium hypochlorite.

Cautions:

- ✓ Use MD-ChelCream with discretion to prevent perforation
- ✓ Avoid extrusion of the MD-ChelCream into the apex
- ✓ Keep the syringe capped all the time
- ✓ Store in temperature 10°C ~ 25°C (50°F ~ 77°C)

Shelf Life:

- ✓ Two years from the production date.