

Hydraulic Temporary Restorative



Advantages:

- ✓ Easy application and removal
- ✓ Hermetically seals the cavity
- ✓ Excellent stability when set
- ✓ Proper crush strength

Components:

- ✓ 40g White or Pink paste in a jar
- ✓ Box size: 38 × 41 × 39(mm)
- ✓ Weight: 55g

Directions:

1. Dry the prepared cavity.
2. Transfer the amounts of MD-Temp required to the cavity by using a suitable instrument.
3. Let the patient bite down on the filling to check for interferences in case of class 1 cavities. Otherwise, be sure to adapt the filling into the cavity walls by contouring it.
4. Remove excess by wiping it off with moist cotton towards the margins of the cavity.
5. As complete setting takes place after an hour, instruct the patient to avoid pressure or abrasion until such time.

1

Shelf Life:

- ✓ Two years from the production date.