# Ezfil

## Dental Composite Resin

**META BIOMED**

2020. 10. 16. DM-162-3

### SYMBOLS

**Manufacturer**

nl Fabrikant  
fr Fabricant  
de Hersteller  
it Produttore  
pt Fabricante  
es Fabricante  
tr Üretici

**Batch Code**

nl Partijcode  
fr Code de lot  
de Chargennummer  
it Codice Lotto  
pt Código de lote  
es Código de lote  
tr Parti Kodu

**Date of manufacture**

nl Productiedatum  
fr Date de fabrication  
de Herstellungsdatum  
it Data di produzione  
pt Data de fabrico  
es Fecha de fabricación  
tr Üretim tarihi

**Use By**

nl Gebruiken vóór  
fr Utiliser avant  
de Verwenden bis  
it Uso previsto  
pt Usar até  
es Usado por  
tr Kullanım

**CE - Mark and identification number of notified body.**  
**1639** Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC

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**Authorized representative in the EC**

nl Erkende vertegenwoordiger in de EG  
fr Représentant autorisé dans l'UE  
de Bevollmächtigter/ in der EU  
it Rappresentante autorizzato nella CE  
pt Representante autorizado na CE  
es Representante autorizado en la CE  
tr AB'de yetkili temsilci

**Manufacturer**

**META BIOMED CO., LTD.**

270, Osongsaengmyeong 1-ro,  
Osong-eup, Heungdeok-gu, Cheongju-si,  
Chungcheongbuk-do, Republic of Korea

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• Ezfil is dental composite resin for Anterior and Posterior Restorations.

**INDICATION**

- Class I, II, III, IV, V restorations
- Core/Buildup to replace missing tooth
- structure
- Diastema closures
- Direct veneers
- Composite and porcelain repairs

**INSTRUCTIONS**

**1. Preparation**

- ① Use it after confirming the shelf life of the product and any abnormal status of the product such as foreign material, impure material, degeneration, etc.
- ② Comply with the use method after familiarizing yourself with the user's manual of the manufacturer.
- ③ Remove the lid and prepare the filling instrument.
- ④ Place the device to protect the airway before performing a procedure.

**2. Use**

- ① Select the shade based on the teeth to be treated. Clean the teeth with water and remove the coloration or plaque. Also, selection should be made by considering the shade and restoration depth.
- ② A cavity must be free from any contamination including saliva or blood. After removing the minimum of the cavity, if the residual amalgam, resin or other base material remains, hardening of the restoration material can be interfered because of blocking the light penetration.
- ③ Etch the entire cavity including the enamel boundary using an acid etchant.
- ④ Apply an bonding agent to the enamel and dentin.
- ⑤ Apply light (wavelength: 420–500 nm, light intensity: 450 mW/cm<sup>2</sup>) to it after restoring the cavity with an adequate amount of the filling material(Ezfil) using a hand instrument.
- ⑥ Repeat the process of ⑤ until the tooth cavity is completely restored.
- ⑦ After restoration with the dental filling material, remove the remaining of the flame-shaped finishing bur and complete the form.
- ⑧ Finish the last polishing using the rubber polishing point and aluminium oxide polishing paste.

**3. Storage and management method after use**

- ① Close the lid tightly after use
- ② Store in a dry and cool place while avoiding direct light and excessive heat.

**CAUTIONS**

**1. General cautions**

- ① Since this product is only used for oral tissues, be careful so that it does not enter eyes. If it enters eyes, wash the eyes with clean water and be examined by an ophthalmologist.
- ② This product cannot be prescribed for internal use.

**2. Warning**

- ① Use of this product for purposes other than the intended purpose or use by non-specialists is prohibited.
- ② This product must be stored at a place where children cannot reach.
- ③ This product shall not be used in patients who have hypersensitive history such as rash, dermatitis, etc.

**3. Cautions for application**

- ① Before use, install the rubber dam in the oral cavity of the patient before procedure, if necessary.
- ② For a patient who develops hypersensitivity reaction such as rash, etc. while using this product, suspend the use and let a

dermatologist examine the patient.

**4. Side effect**

None Known

**5. Contraindication**

Ezfil should not be used if patients are known to be allergic to methacrylate resins.

**STORAGE**

Store at room temperature(15–25°C/59–77°F) while avoiding the high temperature and direct light.

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